

Mounting Description

安装说明

Abduction block

分腿器



Tools: awl, 6mm drill & star screwdriver

工具: 锥子, 6 毫米钻机以及星形螺丝起子



1. Place the abduction block in center and on top of the seat plate, and find the required position for the adapter.

把分腿器放在座板的中央上部, 将连接件放置在要求的位置

Make marks, using an awl, for the screw holes of the adapter, which is to be mounted on the

UNDERSIDE of the seat plate.

用锥子标出分腿器连接件的螺丝孔位置。分腿器连接件应安装于座椅板下侧

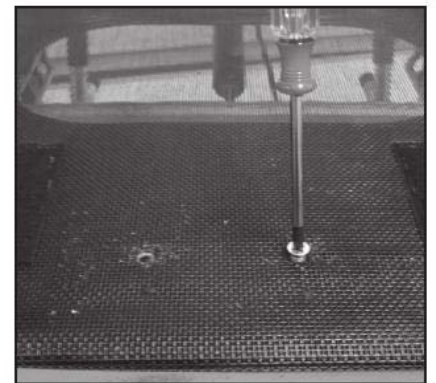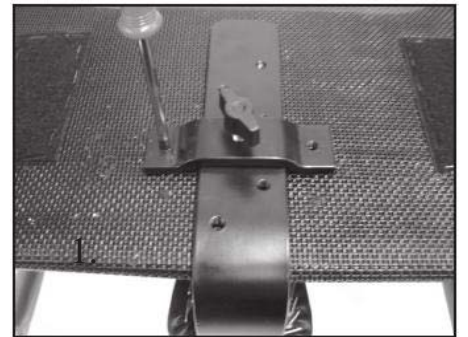

2. Drill two holes through the seat plate. Fix the adapter on the **UNDERSIDE** mounting the screws through the top of the seat plate using a star screwdriver.

在座板上钻出两个小孔。用星形螺丝起子将螺丝从座椅板上穿入后,固定住座椅板下侧的连接件。

3. Mount the abduction block by sliding the bar into the adapter. Fix it by tightening the wing screw.

将分腿器闩推入连接件中,旋紧翼形旋钮将其固定。

Item no.	Item
24297	Abduction block Mini 迷你型分腿器
20860	Abduction block Small 60mm 小号分腿器 60
20861	Abduction block Medium 70mm 中号分腿器 70
20862	Abduction block Large 80mm 大号分腿器 80