Swingable tray with lock

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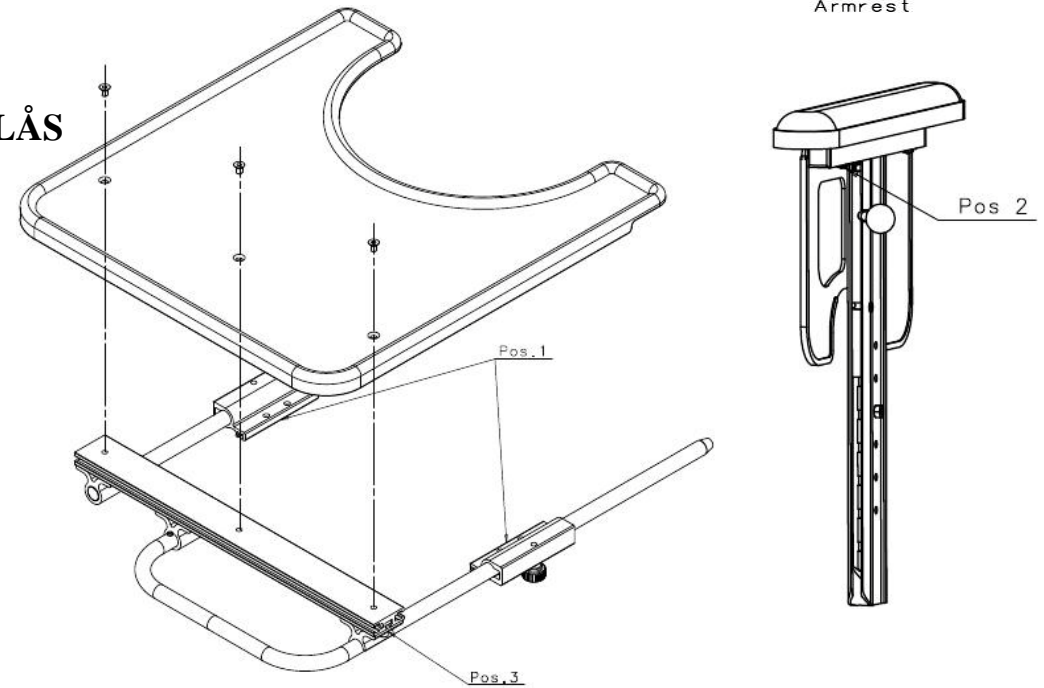
MOUNTING OF SWINGABLE TRAY WITH LOCK

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1. Check that the shipment is according to the order/packing list.
2. 检查包装内物品是否与定货/装箱单一致
2. The front screw (pos 2) that holds the armrest pad should be screwed out. When this is done, the wheelchair is ready for mounting of the universal tray.
松开扶手前端螺丝(件 2), 准备安装万能托盘。
3. The holder (pos. 1) with the star wheels (2 pcs) shall be mounted in the low edge of the armrest. Use the rear hole and mount the armrest pad with machine screw M5 x 35.
带星轮旋钮螺母(件 2)支撑块(件 1)应安装于扶手下端边缘, 用机制 M5 x 35 螺钉将扶手垫安装于扶手后部孔中。

Chair width	Description	Item no:
25-30cm	Universal tray small	15139
35-40cm	Universal tray middle	15140
43-50cm	Universal tray large	15141

4. The tray can now be mounted on the wheelchair. The tubes shall be pushed into the holder (pos. 1) under the armrest. The screws (pos. 3) that hold the tube should not be tightened. Now the tray has to be pushed into the end of the wheelchairs armrest.

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5. To put the tray in the centre position you have to check the distance from the outer edge of the tray to the outside of the armrest pad. The distance must be the same on both sides.

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6. Check that the tray easily slides into the holder. The screws (pos. 3) should be tightened when the tray is in the rear position.

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The tray is now ready for use

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NB! When adjusting to the widest position, the brackets need to be slightly on the outside of the slide bar.

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