

Mounting Description

Side Support Correction

侧部支撑安装说明



Item	Item number
Side support Correction Left	27374
Side Support Correction Right	27375

Sense

Tools: 5mm Allen key

工具: 5MM 内六角扳手

Mount the adapter in required height on the backrest tube. The set screws should be on the inside of the tube.

Fix both screws.

将支架安装在靠背支撑管合适的位置，两个紧定螺丝应置于支架管内侧，并将其固定。

Sense

The adapter is mounted directly onto the backrest tube.

支架直接安装在靠背支撑管上。

Netti models

Un-tighten the straps on the velcro backrest & twist the backrest material around the backrest tube to give room for the adapter. Then adjust the straps as normal.



Netti I II III Swinn All CED

松开靠背尼龙搭扣或靠背管布以便
支架安装.安装完成后,如往常拉紧
靠背尼龙搭扣

NB! When Side Support Correction
is used on the Netti models, we re-
commend to use the Flip backrest
cushion.

请注意! 在 Netti 系列轮椅上使用
侧部支撑板时,我们建议使用薄靠垫.

Lead the side support into the hole in the
adapter, & tighten the adjustment lever.
This is also how to adjust the side support
in & out of the chair.

将侧部支撑置入支架管内,用翼型旋扭扭紧。
拆去侧部支撑: 松开旋扭后取出。

To adjust the side support in the length
of the chair, use an Allen Key.

调节侧部支撑在轮椅上的长度可使用内六角扳手

To adjust the angle of the side support
use an Allen key on the four screws on
the outside of the side support.

调节侧部支撑角度: 用内六角螺丝调节支撑板外侧的四个螺丝

The side support is released by loosening the
adjustment wheel.

取出侧部支撑: 松开翼型旋扭

Then lift the support straight up, and swing it
out to the side.

将侧部支撑提起后将侧部旋转

