

Mounting description for
Recline release handle Netti I-II-III for armrest.
Netti I-II-III 扶手用倾斜调节手柄安装说明


To fix the releasehandle, you must unfix the foremost screw under the armrest pads.

要安装该手柄,先松开扶手垫下最前端的螺丝。

Put a M5x35 screw through the fixing hole on the releasehandle, and mount it under the armrest pad with the existing screw.

将 M5x35 螺丝穿入该手柄的固定孔后，固定在扶手条下方

Fixing hole 固定孔位(右侧)

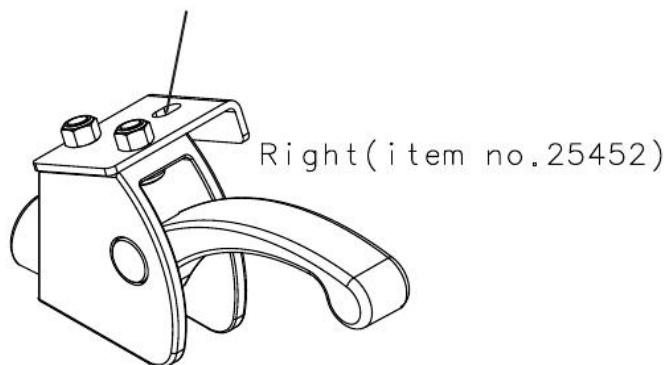

Fixing hole

固定孔位(左侧)

